BioWhiten for Aligners - Instructions for Use

For maximum results, follow the instructions carefully. If you have questions, feel discomfort or to stop treatment, please consult your dentist. Crowns, veneers, fillings, dentures and restorations will not whiten and may require replacement following the whitening procedure.

Note (Before you use BioWhiten for Aligners): If you experience sensitivity, pain, or have any concerns before using this product, contact your dentist. To minimize potential discomfort associated with clear aligner treatment, refrain from using this whitening product for the first two weeks after receiving your initial set of clear aligners and for the first two days following each subsequent aligner change. Patients with amalgam fillings may observe a temporary dark purple discoloration in their trays; this is a normal occurrence and does not affect treatment efficacy.


1. Before whitening, brush your teeth with toothpaste and rinse.
2. Remove the cap of the Whitening Syringe and install the application mixing tip.
3. Apply the gel inside the aligner tray in a continuous bead, filling about one-quarter to one-third of the tray height. Typically, it should be lentil size for each teeth. Apply the gel along the cheek side of the tray for all teeth.
4. Dry your teeth with a clean soft napkin or cloth. If you can, also try to dry your teeth by pulling air.
5. While looking in a mirror, place the Aligner Tray that you have prepared with the gel in your teeth and press it to fit firmly. A small amount of gel may overflow. Clean this overflowing gel with a napkin.
6. The aligner tray should remain in your mouth for at least 45 min or 1 hour. Repeat this process once a day for 10 consecutive days. Each syringe contains enough gel for approximately 4-5 days of treatment. Consult your dentist for the recommended number of treatments.
7. Once finished placing gel in trays, remove mixing tip from syringe and replace with the plastic cap.
8. After each application, Cleanup Rinse and clean the aligner tray with a toothbrush and water. Do not use trays as wet and always dry them before use it
9. Rinse and brush your teeth to remove excess gel.

This product is for professional use only. Warnings & Precautions for Patients:

- If you have allergies and in case of an allergic reaction, rinse thoroughly and seek medical advice.
- Avoid direct contact of the whitening gel with the gums and excessive salivary flow.
- Some users may experience temporary discomfort in the gums, lips, throat, tongue, or teeth. If discomfort persists, discontinue use and contact your dentist.
- Avoid swallowing large amounts of gel. If ingested, drink plenty of water and contact a physician. If aspirated, seek immediate medical attention.
- Pregnant or nursing women should not use whitening products.
- Teeth may re-stain over time. Touch-up treatments may be needed.
- Avoid eye contact; if exposure occurs, rinse immediately.
- Use only with aligners, retainers, or Whitening Trays.
- Do not eat, drink, or use tobacco while wearing the whitening trays.
- Dispose of used syringes and tips properly.

For Dental Professionals:

- Avoid treatment in patients at risk of swallowing or aspirating the tray.
- It is recommended to use this product towards end of each clear aligner replacement periods. e.g. 2 syringes at the beginning of aligner treatment and 1 syringe per every second or third aligner replacement cycle during the entire aligner treatment period.
- Pre- and post-treatment with desensitizing gels, fluoride varnish, or sensitivity relief toothpaste is recommended for patients with known sensitivity.
- Restorative materials such as crowns and fillings will not whiten. Plan whitening treatments before restorative procedures.
- Wait 7-10 days after whitening before placing adhesive restorations.
- Evaluate gingival and oral health before treatment to prevent irritation.
- Avoid enamel etching or polishing before whitening as it may increase sensitivity.
- White spot lesions may become more visible. Continue treatment until color blends, then re-evaluate after two weeks.
- Ensure restorations are sealed and exposed dentin is covered. Use desensitizers or remineralization agents such as BioWhiten NanoCare, if necessary.
- Monitor progress to prevent excessive whitening or translucency.
- Inform patients that some teeth may not whiten effectively and that existing dental work will remain unchanged in color.

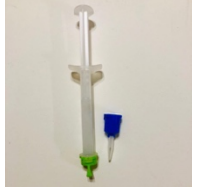CAUTION:

- Do not bend, crush, or deform any material that comes out of your kit. Store the box as you open it.
- Store the Whitening tray and use it to make the whitening again after about 3 months
- Store whitening gels in the refrigerator but do not freeze in the freezer. Store the gel at 2 ° C - 10 ° C (36 ° F to 50 ° F).
- Keep BioWhiten whitening gel for 15-20 minutes at room temperature before use. Keep it in the refrigerator again after use.
- Keep out of reach of children.
- If you experience sensitivity or have any concerns, contact your dental professional.

This product is designed for use with clear aligners, retainers, or Whitening Trays. If a serious reaction occurs, report it to the manufacturer and local authorities.

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